



READY. Set. Go

Simply set in place and secure. No hydration required.

Use When:

- Antibacterial properties are needed to help prevent infection
- Absorption of moderate to heavy levels of exudate is required
- Compression therapy is needed
- Total contact casting is needed

Application:

- Simply place and secure; no hydration required
- Secondary dressing not required
- Tape or wrap required for fixation

Step 1. Select Size and Apply




Select the appropriate dressing size to ensure the dressing will cover the entire wound.

Use with compression



Hydrofera Blue READY foam can be used in conjunction with compression wraps and total contact casting.

Step 2. Cover

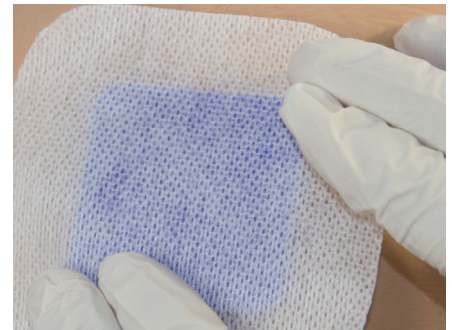

Secure Hydrofera Blue READY with a fixation dressing (eg. adhesive tape).

Wear Time



Dressing can be left in place up to 7 days, depending on the exudate level and clinical conditions of the wound.

Hydrofera Blue READY foam Ordering Information

Product No	Product Size	Quantity/Box
HBRS2520	6.4 cm x 6.4 cm (2.5" x 2.5")	10
HBRS4520	10.2 cm x 12.7 cm (4" x 5")	10
HBRS8820	20.3 cm x 20.3 cm (8" x 8")	10

For more information, see product instructions included with the product.

Hydrofera Dressings
manufactured by
Hydrofera, LLC,
340 Progress Drive
Manchester, CT
06042, USA
www.hydrofera.com