

Application

Step 1

- Cleanse the wound in accordance with established procedures
- Carefully dry the periwound skin
- Select and apply the appropriate dressing size to ensure the dressing will cover the entire wound
- NO hydration is required



Step 2

- Apply an absorptive secondary dressing (e.g. gauze, bordered foam)



Step 3

- Dressing can be left in place up to 7
- Can be used in conjunction with compression wraps and total contact casting



Hydrofera Blue READY-Transfer foam Ordering Information

Stock No	Product Size	Quantity/Box	HCPC
HBRT2525	2.5" x 2.5" (6.4 cm x 6.4 cm)	10	A6209
HBRT4050	4" x 5" (10.2 cm x 12.7 cm)	10	A6210
HBRT8080	8" x 8" (20.3 cm x 20.3 cm)	10	A6211

For more information, see product instructions included with the product.

For product questions, sampling needs, or detailed clinical questions concerning our products in the US call **860-337-7730**.

Hydrofera Blue and the Hydrofera Blue logo are trademarks of Hydrofera, LLC.
©2018 Hydrofera, LLC.

Hydrofera Blue Dressings
Manufactured By:
Hydrofera, LLC
340 Progress Drive
Manchester, CT 06042 USA

www.hydrofera.com